

Hill-Rom®

P500 Therapy Surface



At a glance

Easy To Use:

- Intuitive controls packaged in a lightweight control box
- Quick set-up
- Designed to fit flat deck frames
- Cleaning – wipeable and launderable
- Quiet!

Patient and Caregiver Safety:

- Audible and visual Bed Exit, 30 degree Head-of-Bed and RemindMe™ alarms
- Turn Assist with “confirm rails up” feature
- Control panel lockout function
- X-ray cassette sleeve
- nano Ag+ technology to protect the surface from stain and odor-causing bacteria

Effective Therapy:

- Weight-based, multi-zoned pressure redistribution surface for patients up to 500 lbs
- Advanced Microclimate™ Technology → the next generation low air loss
- Shear and friction management using shear-resistant material and a real-time pressure redistribution algorithm

Easy, Safe and Effective Therapy


The P500 Therapy Surface is designed to provide you with an easy to use, safe surface for effectively treating your patients.

Designed for safe skin and patient comfort by helping to prevent and treat Stage I-IV pressure ulcers.

Hill-Rom®

Enhancing Outcomes for Patients and Their Caregivers.™

Hill-Rom® P500 Therapy Surface

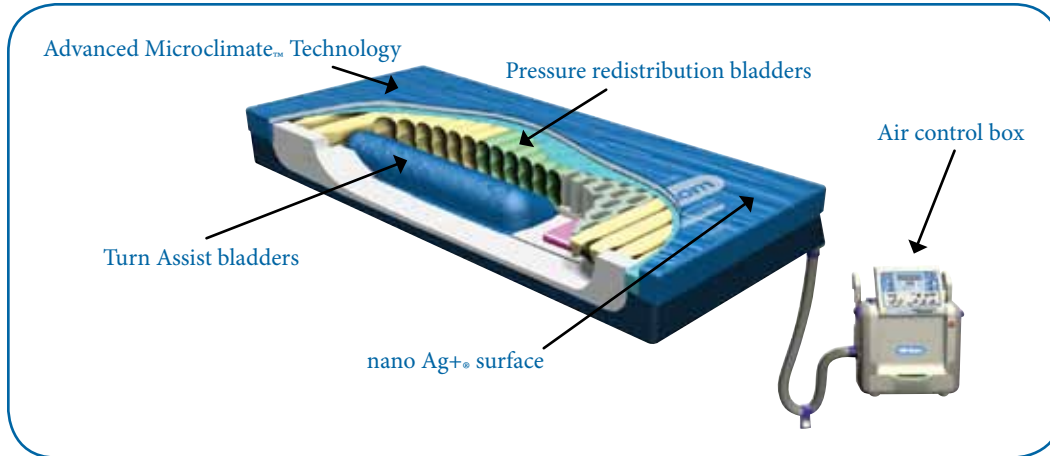

ETL CLASSIFIED

118841

MEDICAL ELECTRICAL EQUIPMENT CONFORMS TO UL Std. 60601-1, IEC 60601-1-4 and CERTIFIED TO CAN/CSA Std. C22.2 NO. 601.1

Comfort Adjust used to optimize patient comfort within the therapeutic range.

Message Center displays surface functions and service mode.

Alarm Silence used to mute all audible alarms.

Normal Mode used for optimal pressure redistribution.

Max Inflate causes surface to firm for repositioning activities.

Lockout key locks all controls on the panel, an additional level of patient safety.

Turn Assist, with Confirm Rails Up safety feature, aids the caregiver in turning the patient left or right.

Enter patient weight to set optimal pressure redistribution. Lb/kg key included to change reading.

Single Mode Patient Out of Bed Alarm to notify caregiver of patient activity.

30° Head-of-Bed Alarm so caregiver can raise or lower head to optimize therapy.

Alarm Volume used to control all audible alarm settings.

RemindMe™ Settings used to alarm at various time intervals for routine patient care tasks.

Technical Specifications

Mattress		
L x W x D.....	84" x 35.5" x 8"	
Weight.....	45 lbs	
Therapeutic weight limit.....	500 lbs	
Air Supply Unit		
L x W x D.....	13.75" x 14.5" x 8.5"	Power Supply..... 120 VAC +/- 10% @ 60 Hz single phase
Weight.....	25 lbs	Operating temperature range..... +15°C to 40°C
		Relative humidity range..... 10% to 90%
		Storage temperature range..... -40°C to +70°C
		Relative humidity range..... 10% to 95%
		Transport inflation time..... 3 hours minimum

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

© 2010 Hill-Rom Services, Inc. ALL RIGHTS RESERVED.



Enhancing Outcomes for Patients and Their Caregivers.™

USA 800-445-3730
Canada 800-267-2337

www.hill-rom.com