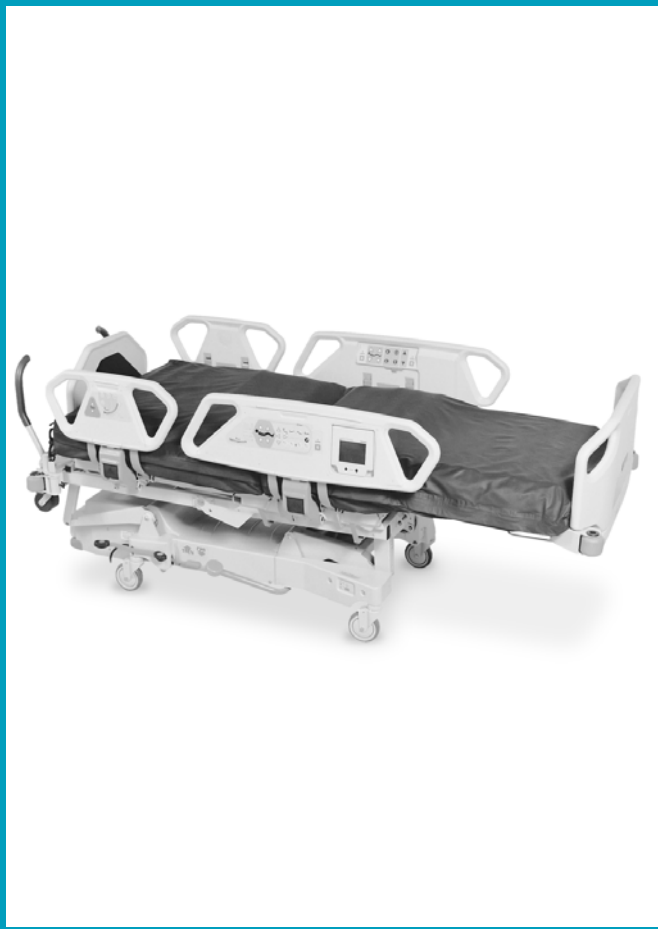
**User Manual Addendum**  
**Ergänzung zur Gebrauchsanweisung**  
**Aanvulling op de gebruikershandleiding**

**TotalCare® Duo2 System**

**Hill-Rom®**  
A HILLENBRAND INDUSTRY

**W13001**

**English****Manufacturer:**

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Cedex 5. FRANCE

Hill-Rom Industries is certified to EN46001, ISO 13485  
and ISO 9001

www.hill-rom.com

W13001/July, 2003

Hill-Rom® is a registered trademark of Hill-Rom  
Services, Inc.

TotalCare® is a registered trademark of Hill-Rom  
Services, Inc.

Duo® 2 is a registered trademark of Hill-Rom  
Services, Inc.

Deteq® is a registered trademark of Hill-Rom  
Services, Inc.

Protex™ is a trademark of Hill-Rom Services, Inc.

No part of this text shall be reproduced or transmitted in  
any form or by any means, electronic or mechanical,  
including photocopying, recording, or by any  
information or retrieval system without written  
permission from Hill-Rom Services, Inc. (Hill-Rom).

The information in this manual is confidential and may  
not be disclosed to third parties without the prior  
written consent of Hill-Rom Services, Inc. (Hill-Rom)

Hill-Rom Services, Inc. reserves the right to make  
changes without notice in design, specifications, and  
models. The only warranty Hill-Rom makes is the  
express written warranty extended on the sale or rental  
of its products.

To order additional copies of this manual, call your  
national Hill-Rom representative (see back cover), and  
place a parts order for part number W13001.

© 2003 by Hill-Rom Services, Inc. ALL RIGHTS  
RESERVED.

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## English

### Symbol Definition

This manual contains different typefaces and icons designed to improve readability and increase understanding of its content. Note the following examples:

- Standard text—used for regular information.
- **Boldface text**—emphasizes a word or phrase.
- **NOTE:**—sets apart special information or important instruction clarification.
- The symbol below highlights a **WARNING** or **CAUTION**:

#### Warning and Caution

- A **WARNING** identifies situations or actions that may affect patient or user safety. Disregarding a warning could result in patient or user injury.
- A **CAUTION** points out special procedures or precautions that personnel must follow to avoid equipment damage.

- The symbol below highlights a **CAUGHT HAZARD WARNING**:

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#### Beknellingsgevaar

**English**

- The symbol below highlights a CHEMICAL HAZARD WARNING:

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## English

### Intended use

TotalCare® Duo2 System is a therapeutic support specifically designed to help prevent and treat wounds, especially pressure ulcers from stage 1 to 4 including surgical grafts in high risk patients.

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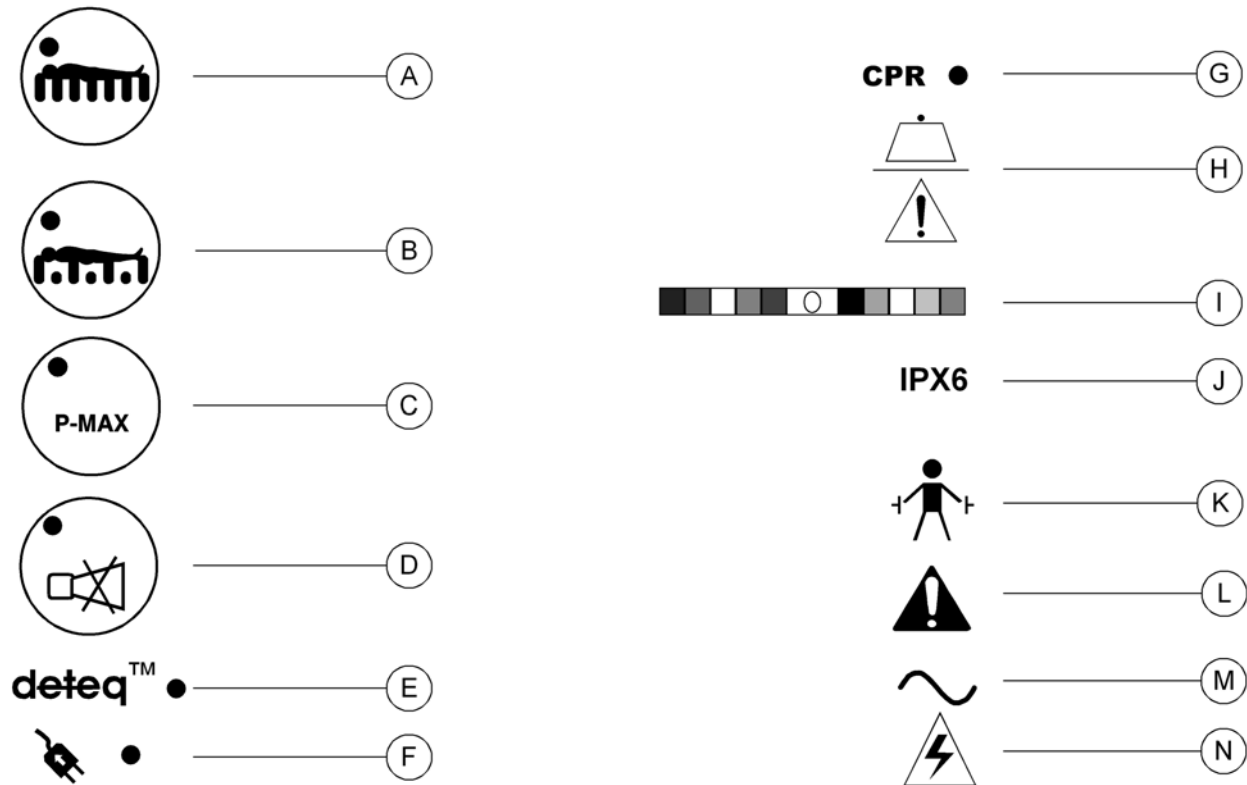
**English****Symbol Quick Reference**

A	Continuous low pressure mode symbol/button
B	Alternating low pressure mode symbol/button
C	Maximum inflation mode button
D	Alarm silence (10 minutes)
E	Deteq® Sensor status
F	Transport mode
G	Cardio Pulmonary Resuscitation Indicator
H	Safe working load: Maximum load on the bed including patient, mattress and accessories that can be permitted according to the manufacturer's declaration
I	Label, colour coded hose connections
J	Jet-proof equipment (equipment protected against powerful water jets)
K	Defibrillation-proof type B applied parts
L	Attention consult accompanying documents
M	Alternating current
N	Dangerous voltage

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**English****Introduction**

The TotalCare® Duo2 System provides two clinically proven therapies combined in one clinically accepted product.

At the touch of a button, you can select either alternating low pressure therapy or continuous low pressure therapy to suit the changing clinical needs of your patient.

Whichever mode you select, the therapeutic mattress will automatically regulate cushion pressures at the lowest possible support pressure. This is due to the patented Deteq® Sensor, which constantly monitors and adapts cushion pressures to each individual patient's line of flotation. Therefore, whatever the size, weight, or bed position of your patients, you are giving them the most comfortable and effective therapy available, automatically.

Our prevalence survey experience with over 50,000 patients across Europe shows that in 31% of patients, the patient's heels are particularly at risk of skin breakdown. TotalCare® Duo2 System incorporates an ultra low pressure heel zone in both continuous and alternating therapy modes, designed to treat as well as help to prevent skin damage in this extremely vulnerable area.

All of these advanced functions are selected via the head siderail outer panel. Also selectable at this panel is maximum inflation mode, alarm silence and CPR rapid deflation and bed flattening is a one step action activated from the CPR Pedal.

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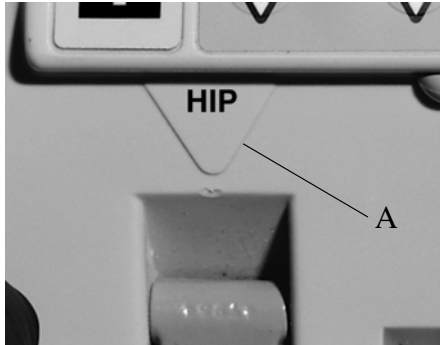
## English

### Therapy Modes

1. When AC power is connected to the bed, the sleep surface will inflate automatically and the active therapy mode will be indicated on both head siderails.
2. In the case of short patients, centre the patient's sacrum at the Deteq® Sensor zone marked on the side of the mattress cover and/or with the hip locator (A) on the intermediate siderail.

#### NOTE:

The head siderail will display the status of the mattress continuously.

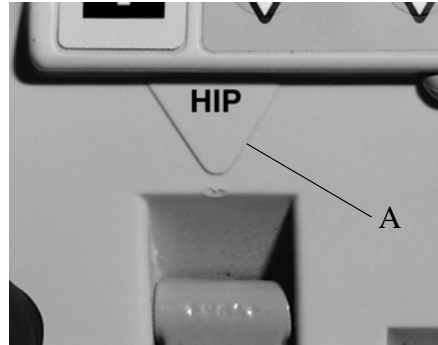
### Ultra Low Pressure Heel Zone

The treatment surface also provides optimal pressure relief for the patient's heels. This zone is active in both continuous and alternating therapy modes when the foot section is either retracted or fully extended, and when the bed is either flat or in the chair position.

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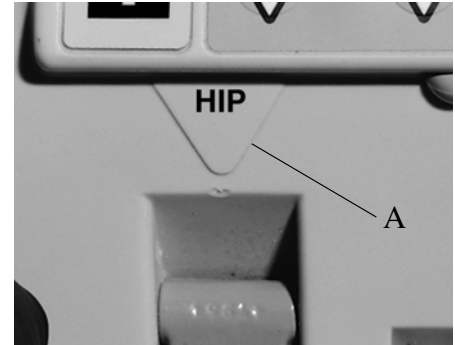
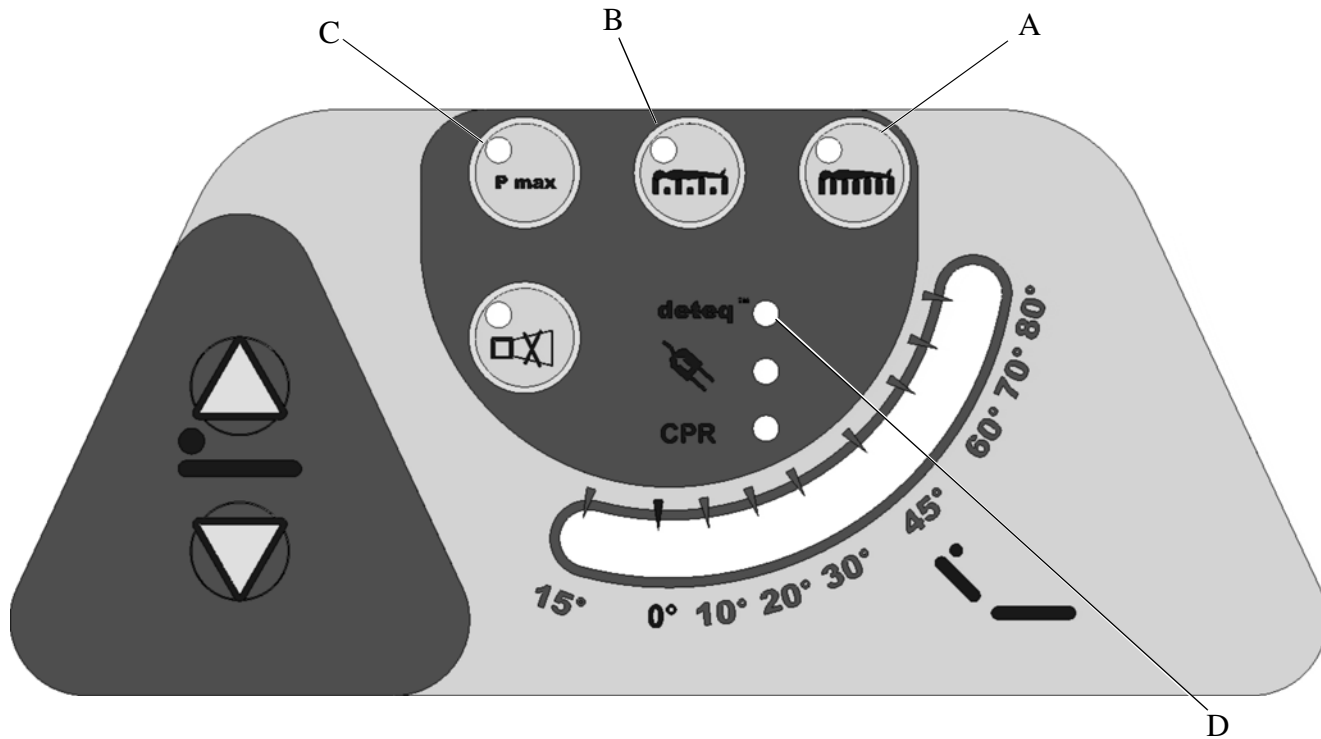
### Instructions for Use

#### Continuous Low Pressure Mode

To select this mode, press the continuous low pressure button (A) and the corresponding LED illuminates green. The patient is supported at optimum low pressure under the control of the Deteq® Sensor system.

#### Alternating Low Pressure Mode

To select this mode, press the alternating low pressure button (B) and the corresponding LED illuminates green. The patient is supported at optimum low pressure under the control of the Deteq® Sensor and the cushions deflate sequentially in a 4 step cycle of approximately 10 minutes. The heel zone also sequentially deflates and the cushions inflate slightly to produce pressure changes in a 2 step cycle under the patients heels.

#### Maximum Inflation Mode

To select this mode, press the P.Max button (C). The corresponding LED illuminates yellow. This mode will make the patient support surface very firm, and therefore will no longer have a therapeutic effect for the patient. An alarm sounds automatically, and once silenced will resound after 10 minutes to remind you that this non-therapeutic mode is in use, see "Alarm Silence" on page 17 to silence the alarm permanently. In addition, the Deteq® Sensor LED (D) will change from green to red to indicate that the low pressure control has been overridden.

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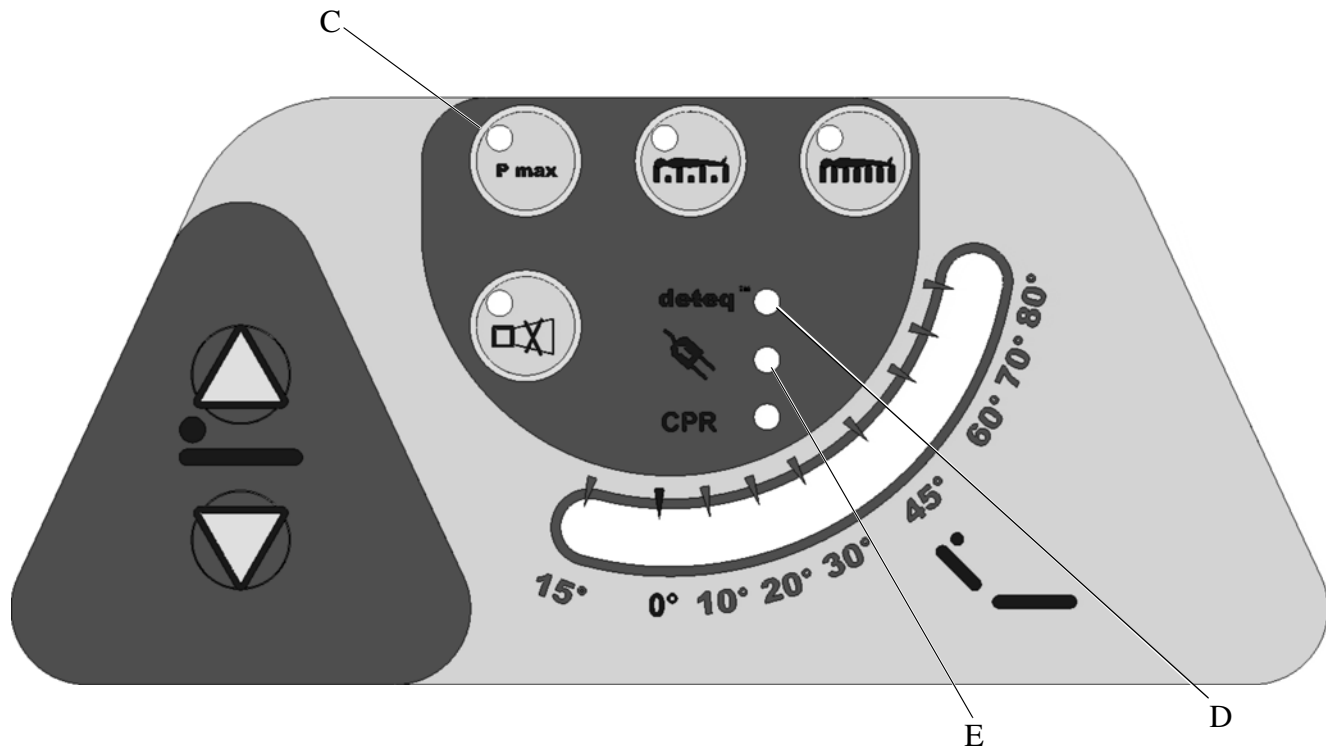
## English

### Transport Mode

The treatment surface has been designed to remain inflated for more than 2 hours without power to allow you to maintain therapeutic support during patient transport. When transporting the patient on the bed, disconnect the power cord from the wall outlet.

#### WARNING

In transport mode, a backup battery provides power for functions such as CPR. If the battery is discharged, the unit must be connected to a wall outlet to charge the battery. Failure to do so could result in the inability to engage CPR in transport mode.

At the loss of power, the Deteq® Sensor LED (D) will go out, the transport mode LED (E) will be illuminated yellow to indicate the interruption to power. In addition, an alarm will sound, and once silenced will resound after 10 minutes to remind you to reconnect the power cord, see “Alarm Silence” on page 17 to silence the alarm permanently.

### Deteq® Sensor Indicator

The LED next to the Deteq® Sensor symbol (D) indicates the status of the system that controls low pressures in both continuous and alternating therapy modes. When the LED is green, the Deteq® Sensor is working automatically.

When the LED turns red, this indicates either that P.Max (C) is in use or that there is a malfunction. See “Troubleshooting” on page 21.

When the LED goes out, there is no longer any power supply.

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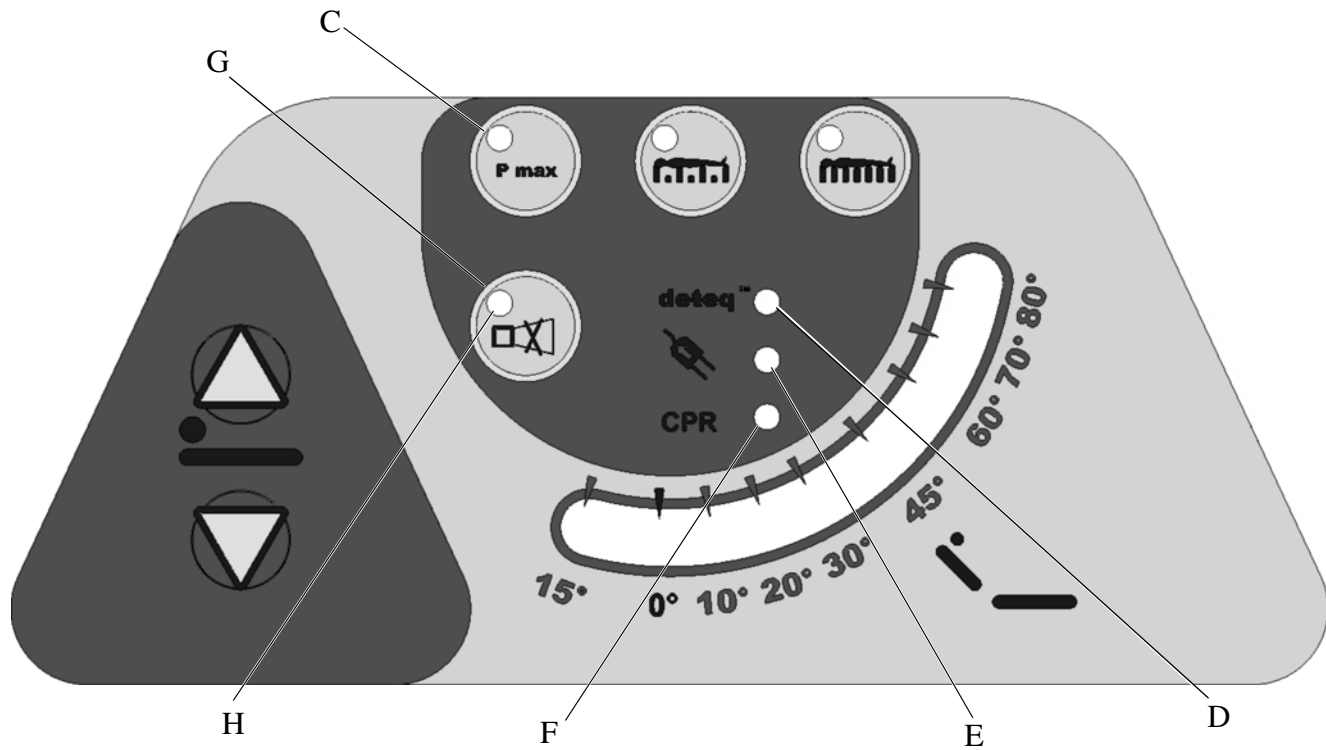
## English

### Alarm Silence

As described, an alarm will sound when CPR, PMax (C), or transport modes (E) are being used. To silence the alarm, press the alarm silence button (G).

The LED (H) will illuminate red to continuously indicate that an alarm is enabled. The alarm will resound after 10 minutes.

The alarm can continue to be silenced for time periods of 10 minutes until either function is disabled.

If, for example, transport mode is required for an extended time (greater than 10 minutes), the audible alarm can be silenced permanently by pressing the alarm silence button until the alarm silence LED (H) flashes.

#### NOTE:

The cause of the alarm must be resolved in order to cancel the visual alarm. For example, plug the unit into an appropriate power source following transport mode.

If there is any malfunction, this alarm will sound. Silence the alarm in the same way, and refer to “Troubleshooting” on page 21.

### CPR Mode

When connected to AC power, the CPR release lowers the head and knee sections, and raises the foot section whilst deflating the treatment surface. The head section moves to a flat position within 10 seconds and the treatment surface deflates in approximately 15 seconds (depending on the patient's weight) to provide a firm base for CPR and the CPR LED (F) will illuminate. Emergency CPR is also functional in the chair egress, or recliner positions. When the pedal

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I

## English

is held down for 4 seconds, a tone sounds and the foot sections rises. The foot section moves to a flat position within a maximum of 25 seconds if fully articulated. If the power cord is unplugged, only the head section lowers. To stop the automatic foot up articulation, press any bed control except for Bed Up/Down, and the foot section stops.

The headboard can be used as a CPR board in emergency situations.

### To Activate CPR:

- If the master lockout is enabled, it must be deactivated to allow other controls to stop the foot section.
- Hold the CPR pedal (I) down with your foot until the head section reaches the flat or desired position and you hear the audible tone. Release the CPR pedal (I) to stop head section movement. The foot and knee sections will automatically move to a flat position.
- The treatment surface automatically deflates.
- To stop foot section movement, simply press any siderail control.

### NOTE:

When AC power is lost, only the head section will lower but the treatment surface will deflate automatically.

### NOTE:

The Bed Up/Down controls are usable when the CPR function is activated.

Following CPR procedures, select a therapeutic mode to inflate the treatment surface.

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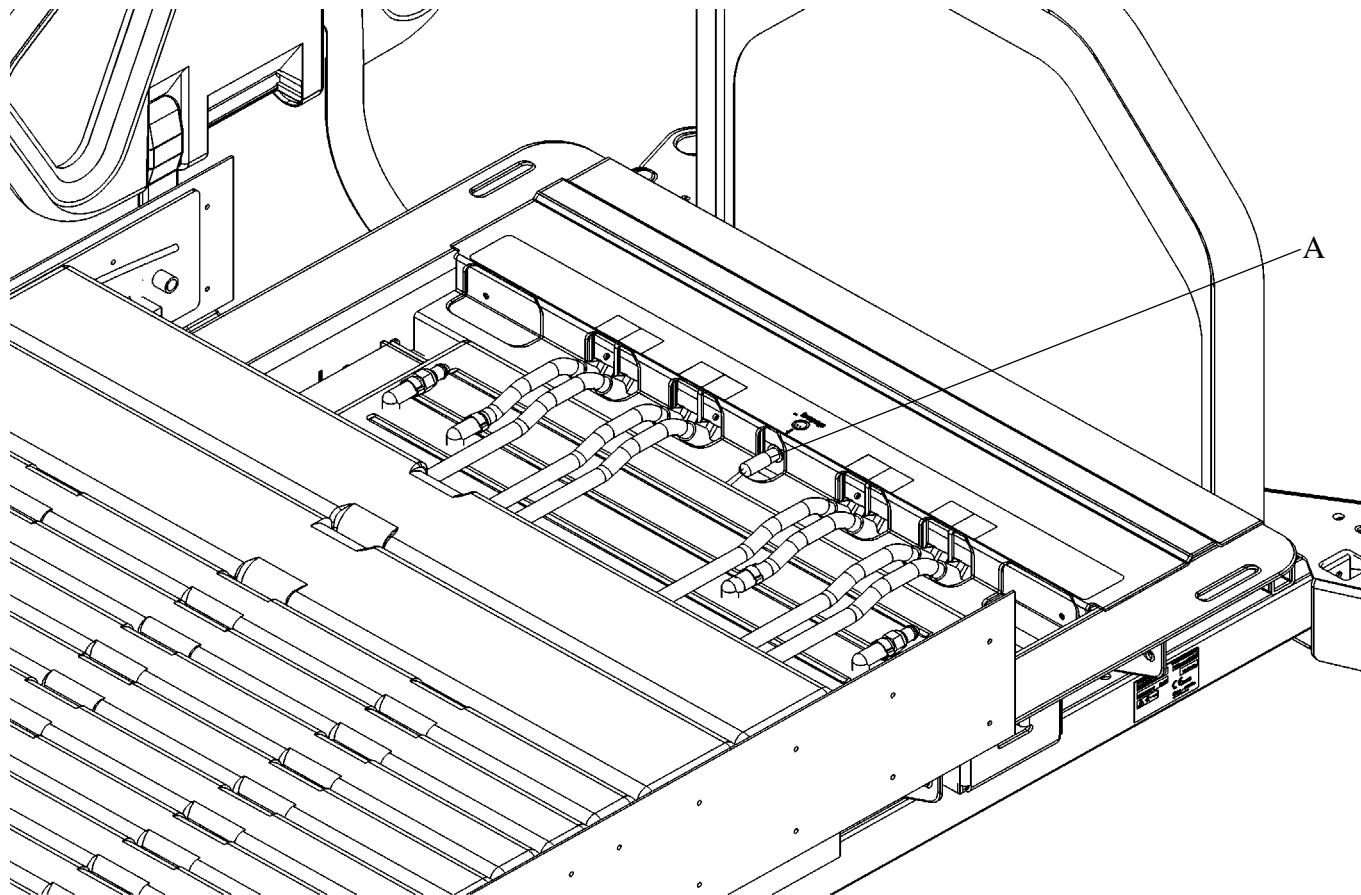
## English

### Troubleshooting

Incorporating the most advanced technology available, TotalCare® Duo2 System is designed to function totally automatically. Therefore, troubleshooting of the unit will be limited to only a few possible areas.

In the event of a Deteq® Sensor alarm, press the alarm silence button to stop the audible alarm.

- Check for tears or large punctures in the mattress.
- Check that the Deteq® Sensor cable (A) has not become damaged or disconnected.
- Check that the Deteq® Sensor foil is located correctly and that the seven snaps on three sides are securing the foil to the base of the Deteq® Sensor module.
- Check the cleanliness of the air filter. This should be cleaned using a vacuum cleaner at least once a year.

If no damage is found, but the alarm persists, contact your Hill-Rom representative for service.

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
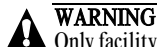
**English****Maintenance****WARNING**

Only facility-authorized personnel should perform preventative maintenance on the TotalCare® Duo2 System. Preventative maintenance performed by unauthorized personnel could result in personal injury or equipment damage.

**Service Calls**

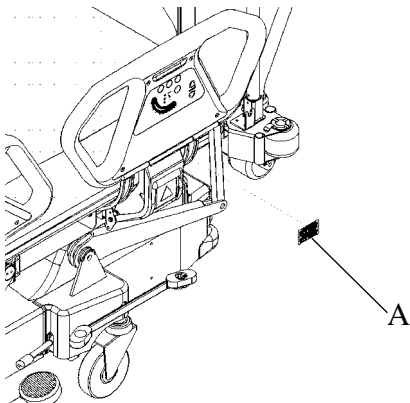When you call Hill-Rom about your unit, be prepared to give the serial number from the product identification label. The product identification label (A) is located at the head left below the mattress as shown in the illustration below.


When you give the serial number, the Hill-Rom representative can identify your unit and give you the information you require more quickly.

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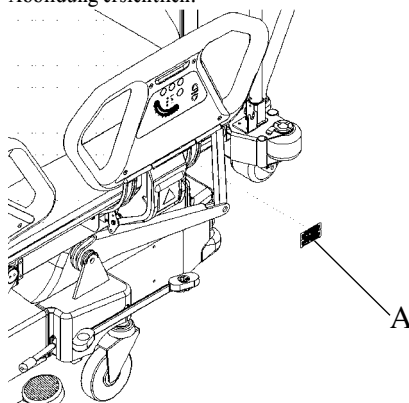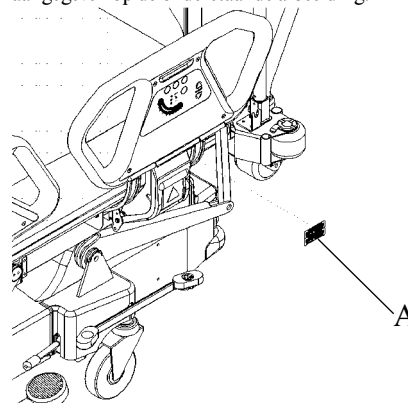
## English

### General Cleaning

 **WARNING**  
Follow the product manufacturer's instructions. Failure to do so could result in personal injury or equipment damage.

 **SHOCK HAZARD**  
Unplug the unit from its power source. Failure to do so could result in personal injury or equipment damage.

 **SHOCK HAZARD**  
Do not expose the unit to excessive moisture. Personal injury or equipment damage could occur.

 **CAUTION**  
Avoid the use of cleaning solutions containing Dimethyl formamide, Tetrahydrofuran, Phenol, Cresol or concentrated inorganic acids, as these chemicals will damage the cushion fabric.

**NOTE:**  
These chemicals are not normally found in standard hospital cleaning solutions.

 **CAUTION**  
Avoid highly alkaline and strong organic based detergents and disinfectants, as these chemicals may attack the cushion fabric dye, causing transfer of colour.

Clean the unit with a lightly dampened cloth and ordinary disinfectants.

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**English****Cleaning Hard to Clean Spots**

To clean difficult spots or stains, use standard household cleaners and a soft bristle brush. To loosen heavy, dried-on soil, you may first need to saturate the spot.

**Disinfecting****⚠ CAUTION**

The mattress is protected against a jet of water from a standard nozzle of 12.5 mm diameter at a distance 2.5 m to 3.0 m, with a delivery rate 100 l/min, spraying the mattress from all practical directions. Check the capability of the cleaning equipment prior to using it, otherwise damage to the mattress may occur.

**⚠ CAUTION**

Do not use hypochlorite (bleach) above a concentration of 500 ppm as this will discolour and weaken the fabric.

When there is visible soilage and also between patient use, we recommend that you disinfect the unit using a tuberculocidal disinfectant.

Dilute the disinfectant as specified on the manufacturer's label and follow the manufacturer's instructions for "wetting" times for hard surface disinfection.

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### Laundering

#### Therapeutic Mattress and Cover

**⚠ CAUTION**  
Tube connections under the therapeutic mattress must be sealed (seals sold separately) prior to laundry. Failure to do so could result in equipment damage.

**⚠ CAUTION**  
Do not use hypochlorite (bleach) above a concentration of 500 ppm as this will discolour and weaken the fabric.

We recommend that mattress cushions & Protex™ Covers should be laundered at 67°C for a minimum of 10 minutes.

**⚠ CAUTION**  
If a tumble dryer is used; do not use heat above 67°C as this may damage the urethane lining and heat seals of the cushions.

Following laundry, tumble dry on cool setting.

#### Routine Maintenance

Check the air filter at least once a year and clean with a vacuum cleaner. Replace if necessary.

It is recommended that the Full Service Checklist (see the Service Manual) be completed at least every six months.

Never discard the battery without previously checking the relevant local environmental policy.

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**English****Contraindications in Use****WARNING**

Under normal circumstances, proper alignment on an air bladder system is easy to maintain; however, through user error or equipment malfunction, cushions may deflate, putting proper alignment at risk. The choice of a therapeutic support for such conditions as unstable spinal cord injuries is based upon the medical judgement of professionals, and each case should be evaluated individually otherwise personal injury or equipment damage could result.

The TotalCare® Duo2 System is contraindicated for patients with:

- An unstable spinal injury
- Cervical or skeletal traction

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## English

### Safety tips

Safety hazards may arise due to improper use of the product. The following safety recommendations must be carefully read and applied.

#### **WARNING**

This equipment is intended to be connected to a dedicated power supply (normally supplied through an isolation transformer) as installed in hospital facilities, otherwise personal injury or equipment damage can occur.

#### **WARNING**

It is recommended that when the patient is left unattended, the siderails are always locked in the raised position and the bed is left in the lowest position. Failure to do so could result in personal injury or equipment damage.

#### **CAUTION**

Since the mattress has influence only upon the main extrinsic factors to the development of pressure ulcers, extensive consideration should be given to intrinsic factors like the physiological condition of the patient (age, morphology, administration of nutrition, anaesthetics and other medicinal substances).

#### **SHOCK HAZARD**

The potential for electric shocks exists with electrical equipment. Establish policies and procedures to educate your staff on the risks associated with electrical equipment.

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**English****SHOCK HAZARD**

Always unplug the unit during cleaning or maintenance operations. Failure to do so could result in personal injury or equipment damage.

**WARNING**

Some pacemaker programmers may not function correctly when the TotalCare® Duo2 System is powered. Unplug the unit from its power source while programming the pacemaker. Failure to do so could result in the inability to program the pacemaker.

**WARNING**

Take care when moving the TotalCare® Duo2 System not to run over the Power cord. Failure to do so could result in personal injury or equipment damage.

**WARNING**

Reliance is placed on the caregivers and their professional judgement to assess the safety of the patient when accommodated in a medical system made of TotalCare® Duo2 System, a combination of accessories and other medical devices, as well as any potential hazard arising from his or her physical and/or psychological condition, otherwise personal injury or equipment damage could result.

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### Technical Data

#### Regulatory Requirements

The TotalCare® Duo2 System meets the provisions of the Council Directive 93/42/EEC.

#### Product Classification

The TotalCare® Duo2 System is a class IIa medical device in accordance with the Council Directive 93/42/EEC.

#### Protection type against electric shock

Class I equipment in accordance with EN 60601-1, 1995 edition.

#### Protection degree against electric shock

Defibrillation-proof type B applied parts in accordance with Amendment 2 of EN 60601-1, 1995 edition.

#### Protection of mattress against harmful ingress of water

Equipment protected against powerful water jets. IPX6 in accordance with EN 60529, 1991 edition.

#### Radio disturbance characteristics

Class A equipment in accordance with EN 55011, 1998 edition.

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## English

### EN ISO 10993-10, 1995 edition:

Biological evaluation of medical devices:  
Part 10: tests for irritation and sensitisation.

### EN ISO 10993-5, 1999 edition:

Biological evaluation of medical devices:  
Part 5: tests for cytotoxicity *in vitro* methods.

## Specifications

### Enclosed Cover

The Protex™ Cover is made of fabric which has low friction, stretches two ways, and is breathable. The Protex™ Cover is bacteriostatic, fungistatic and antimicrobial. It can be wiped down and washed.

### Total Patient Support Surface

Height (Inflated)	13.5 cm (5.3")
Width (Inflated)	85 cm (33.5")
Length (Inflated)	182.0 to 212.0 cm (71.7" to 83.5")
Safe Working Load	Refer to the user manual's specifications of the TotalCare® Bed System
Maximum acceptable patient mass with clinical efficacy for any head of bed elevation excluding bed necessities and accessories	150 kg (330 lb)
<b>Therapeutic Mattress</b>	
Material	Nylon/Urethane Reusable/Washable

### NOTE:

All mattress dimensions are approximate.

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### Electrical Characteristics

#### WARNING

Do not operate the equipment in the presence of flammable anaesthetic mixture with air, oxygen or nitrous oxide. Personal injury or equipment damage could occur.

Voltage:	230V AC, 50 Hz
Current:	
rated	650mA
normal operation	350mA
stand-by	300mA
Power consumption:	
normal operation	80 W
Continuous earth leakage current:	
in normal condition	below 500 $\mu$ A
in single fault condition	below 1000 $\mu$ A
Continuous enclosure leakage current:	
in normal condition	below 100 $\mu$ A
in single fault condition	below 500 $\mu$ A
Patient leakage current:	
in normal condition	below 100 $\mu$ A
in single fault condition	below 500 $\mu$ A
Protective earthing	below 0.2 $\Omega$

The therapeutic surface is designed for continuous operation.

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**English****APPENDIX A****Responsibility of the Manufacturer**

The manufacturer cannot be held responsible when:

- Repairs, unit installation, assembly, modification or verification and test activities are not carried out by field service personnel or authorised company personnel.
- The electrical installation does not provide for the operation of medical devices in accordance with EN 60601-1 and, especially for wards, when the electrical installation is not in compliance with the applicable requirements for medical rooms.
- The unit is not used according to this manual.
- Accessories are used that do not comply with requirements of this manual.

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